

!!! WARNING !!!

Important Information for VIASS Pro²

Stand: 03 / 2021 – V1.0



VIASS GmbH.

Ankenreuthe 98 | 6858 Bildstein | Austria
www.viass.at | office@viass.at

Buttons !!! IMPORTANT !!!



Do not press any buttons before the VIASS Pro2 has not started.

The VIASS Pro² has started when the green LED lights up and two PIP tones can be heard in the headphones.

Failure to observe this can lead to a complete loss of the operating system. This error can then only be restored in the factory in Austria.



Firmware

Always make sure that your VIASS Pro² has installed the last firmware update

You can find in this Link:

<http://www.viass.at/download.html>



Instructions for firmware updates

www.viass.at/dok/Firmware_Update.pdf

Lens



The calibration of the lens is changed by opening the lens screws. And the VIASS Pro² no longer works.

The new calibration can only be restored in the factory in Austria

